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(EN) Produkt: BUDDYCARE®MED Sport-Tape (inelastic, latex-free, non-sterile) – 3.8 cm × 10 m. Material: Cotton fabric, acrylic adhesive (latex-free). **Intended purpose:** Inelastic fixation tape for stabilization, relief, and limitation of movement for joints/muscles. Also, for securing dressings. **Users:** Laypersons & professionals from 6 yrs. Use on children only under adult supervision. **Application: Prepare:** Skin must be clean, dry, oil-free. Shorten hair if dense. Protect sensitive skin with underwrap if necessary. **Cut:** Adjust to required length. Round edges for better adhesion. **Apply:** Apply tape smoothly, wrinkle-free, with appropriate tension. Stabilize joints with overlapping strips. Do not wrap continuously. **Check:** Check fit, avoid constriction. Circulation & sensation must be normal. **Duration & Removal:** Typically wear up to 24 hrs. Remove sooner after sports/sweating or if discomfort occurs. To remove, peel off slowly & flat in the direction of hair growth. Moisten with oil/adhesive remover if needed. **Do not use for Contraindications:** Allergy to acrylates/adhesive plasters. Arterial circulatory disorders. Severe sensory disturbances/neuropathies. Highly irritated/pre-damaged/infected skin. Directly on open wounds (only over a sterile dressing). **Warnings & Reactions:** For external use only. In case of redness, itching, blisters, pain, numbness, or a cold sensation **REMOVE TAPE IMMEDIATELY**. For unclear/severe injuries, consult a doctor/physio. Seek medical advice for persistent symptoms. **Product Info & Storage:** Single-use product. Do not reuse. **Storage:** Cool & dry (5–25 °C), protect from sun/moisture. **Disposal:** Via household waste, observe local regulations. **EXP/LOT/UDI:** See packaging. Do not use after the expiry date. **Reporting of serious incidents:** Report to the manufacturer and the competent national authority. **More info (languages, videos, symbols):** www.buddycare-med.com. **Date:** 2025-10. **Versión:** V1.0

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