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**(EN) Product: BUDDYCARE®MED Fingertape (latex-free, non-sterile).** **Material:** Polypropylene, cohesive coating (latex-free; adheres to itself, not to skin/hair). **Intended purpose:** Elastic fixation tape for finger and nail protection; securing of wound dressings; protection against friction, pressure spots, blisters. **Users:** Lay persons and healthcare professionals. **Target group:** adults and children (children only under adult supervision). **Application:** **Prepare:** Clean and dry area; for open wounds, use a sterile wound dressing. **Apply:** One layer over the fingertip, then wrap crosswise from the joint to the fingertip 4–5× in overlapping layers; no closed circular turn over the joint. **Check:** Check fit, avoid constriction; circulation and sensation (fingertip) normal; nail bed visible or check capillary refill. **Secure:** Tear off and press the end down firmly. Do not use as a pressure dressing to stop bleeding. **Duration & Removal:** Change after a few hours, at the latest after 24 h; change immediately after sports/sweating or if soiled/wet. To remove, simply unwind. **Do not use for Contraindications:** Intolerance to components. Arterial circulatory disorders. Severe sensory disturbances/neuropathies. Highly irritated/pre-damaged/infected skin. Directly on open wounds (only over a sterile dressing). **Warnings & possible reactions:** For external use only. In case of redness, itching, pressure pain, numbness, or a cold sensation, remove immediately. Seek medical advice for persistent symptoms. For unclear/severe injuries, consult a doctor. **Product Info & Storage:** Single-use product, do not reuse. **Storage:** cool and dry (5–25 °C), protect from sun/moisture. **Disposal:** Household waste, observe local regulations. **EXP/LOT/UDI:** see packaging. Do not use after the expiry date. **Reporting of serious incidents:** Report to the manufacturer and the competent national authority. **More info (languages, videos, symbols):** www.buddycare-med.com. **Date:** 2025-10. **Versión:** V1.0

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